

RelyX™ Ultimate

Adhesive Resin Cement



Choose ultimate bond strength,
get ultimate simplicity.

Now, ultimate bond strength can be yours with **FEWER** working steps!

The toughest challenges have been met by RelyX™ Ultimate Adhesive Resin Cement and Scotchbond™ Universal Adhesive. Together they deliver ultimate bond strength, coupled with ease of use.



It's the ultimate solution in dental cements.

RelyX Ultimate cement in combination with Scotchbond Universal adhesive is indicated for final cementation of:

- CAD/CAM restorations
- Glass ceramic restorations
- All-ceramic, composite or metal inlays, onlays, crowns and bridges; 2-3-unit Maryland bridges and 3-unit inlay/onlay bridges*
- Resin nano ceramic
- All-ceramic or composite veneers
- All-ceramic, composite or metal restorations to implant abutments
- Posts and screws

*Excluded for patients with bruxism or periodontitis.

A resin cement ideally suited for CAD/CAM and glass ceramic restorations.

RelyX™ Ultimate Adhesive Resin Cement is an innovative dual cure, adhesive resin cement designed to meet the specific needs of CAD/CAM and glass ceramic cementation. Here's why:











- Industry-leading bond strength
- Fast and easy—only two components (automix syringe and bottle)
- Versatile—for total-etch, selective-etch or self-etch procedures
- World-class esthetics with natural fluorescence
- Ideal for CAD/CAM and glass ceramic restorations



Superior color stability for lifelike esthetics.











RelyX Ultimate cement is free of aromatic amine components to ensure color stability. Compared to other leading cements, RelyX Ultimate cement has excellent color stability and lifelike fluorescence.



Color Stability	Before	Coffee
RelyX™ Ultimate Adhesive Resin Cement		
Panavia™ F 2.0		
NX3		
Multilink® Automix		
Variolink® II		

Resistance to staining: RelyX™ Ultimate Adhesive Resin Cement shows least discoloration after a 3-day incubation in coffee solution.

Source: 3M ESPE internal data

Fluorescence	10% UV	90% UV
Human Tooth		
RelyX™ Ultimate Adhesive Resin Cement		
Panavia™ F 2.0		
NX3		
Multilink® Automix		


Fluorescence: RelyX™ Ultimate Adhesive Resin Cement has fluorescence more similar to a natural tooth than other resin cements.

Source: 3M ESPE internal data

Ultimate bond strength

Take bonding confidence to a new level.


RelyX Ultimate cement delivers high immediate bond strength to dentin and enamel.

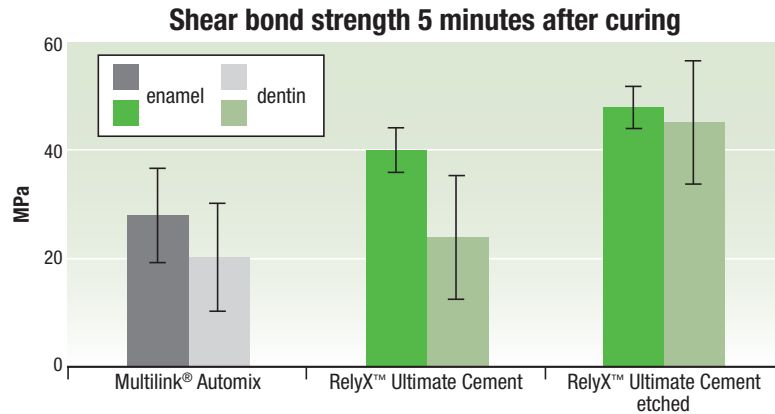


Test: Shear Bond Strength testing. RelyX™ Ultimate Adhesive Resin Cement was used with Scotchbond™ Universal Adhesive (no separate light cure), Multilink Automix with primer A&B according to manufacturers' instructions. All cements were light cured.

Source: 3M ESPE internal data

RelyX Ultimate cement delivers high bond strength to dentin within 24 hours and long term.

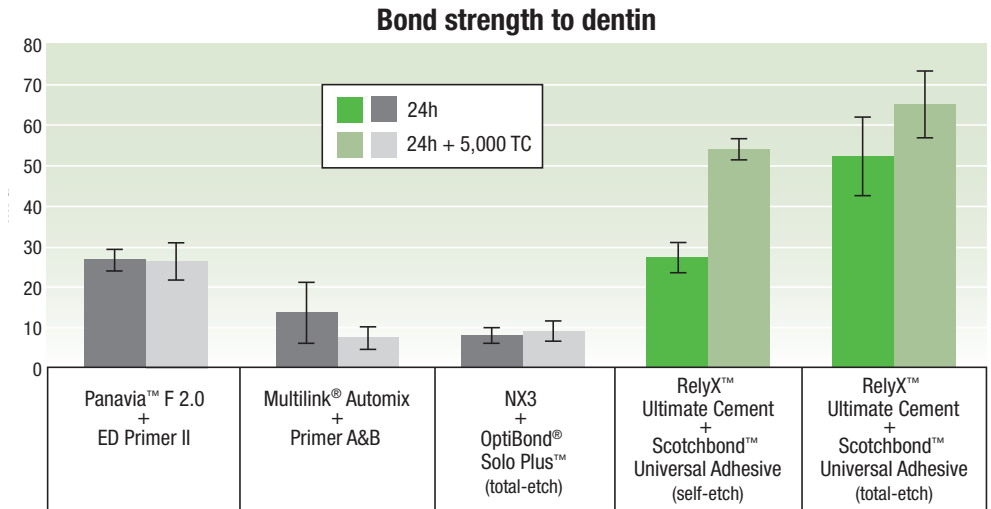

RelyX Ultimate cement delivers high bond strength to enamel at 24 hours and long term.

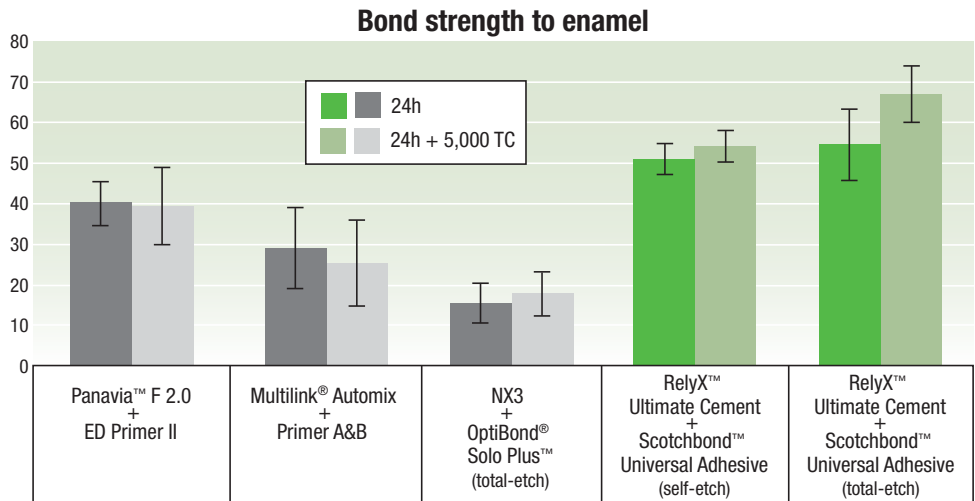

Test: Shear Bond Strength testing. RelyX™ Ultimate Adhesive Resin Cement was used with Scotchbond™ Universal Adhesive (no separate light cure), the other materials with their respective adhesive systems on bovine enamel according to manufacturers' instructions. All cements were light cured. Artificial aging: 5,000 thermocycles (5-55°C).

Source: Wiedig, et al. IADR CED 2011, #430.

Ultimate simplicity

The resin cement solution that delivers more of what you need ...
a fast and easy procedure with fewer components.


Benefits of RelyX™ Ultimate Adhesive Resin Cement:

- Dual cure cement with ultimate bond strength
- Ease of use (integrated dark cure activator for Scotchbond Universal adhesive)
- Tooth-like fluorescence and high wear resistance
- Matching try-in pastes


Benefits of Scotchbond™ Universal Adhesive:

- Combined total-etch and self-etch bonding capability
- Integrated primers for glass ceramics, oxide ceramics and metal
- Consistent bond strength to both moist- and dry-etched dentin
- Virtually no post-op sensitivity in total-etch and self-etch modes

2 components are all you need.

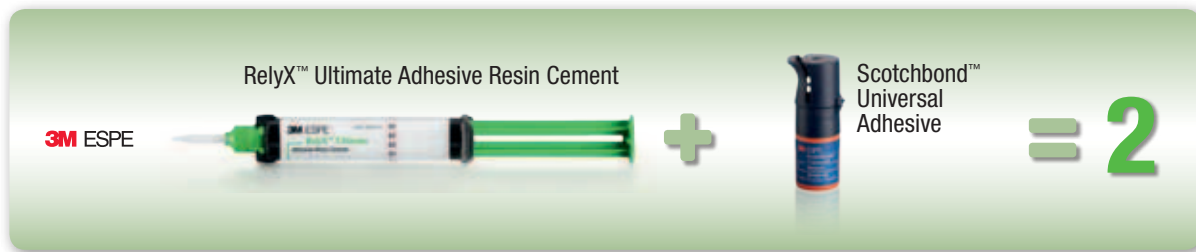

Compare that to the complexity of other leading dental cements:

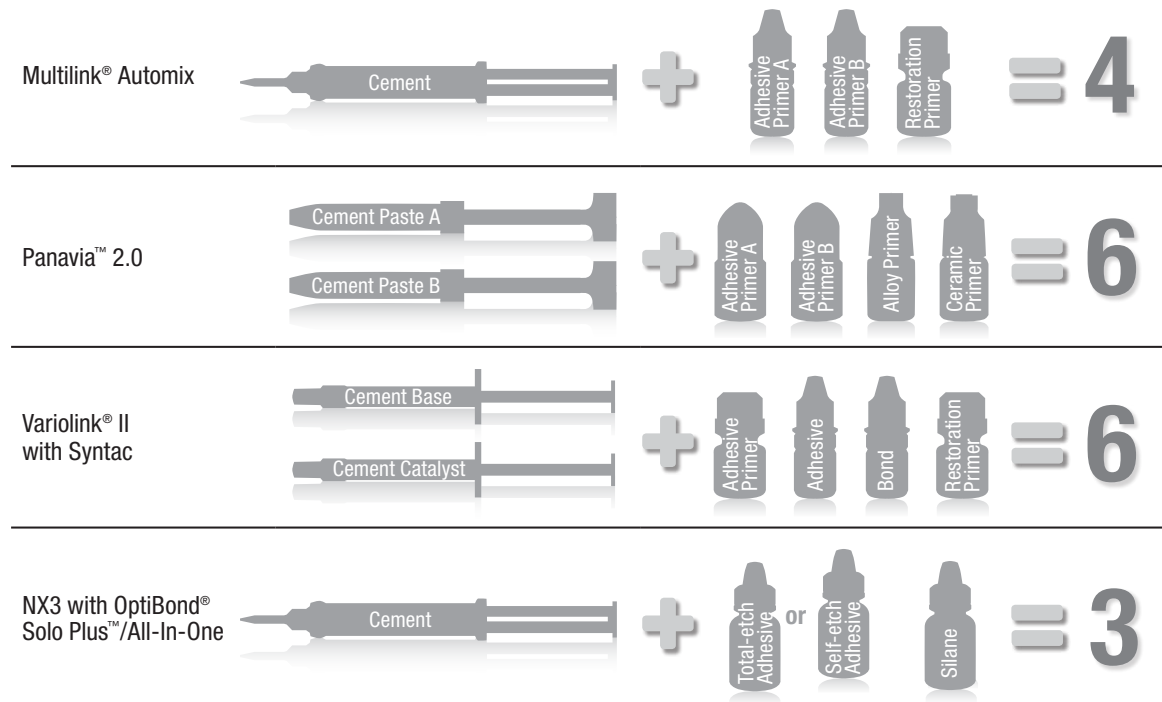

Ordering Information

Item No. Product Information

RelyX™ Ultimate Adhesive Resin Cement

56891	RelyX™ Ultimate Introductory Kit (incl. 4 color shades) 1 – 5ml bottle Scotchbond™ Universal Adhesive; 1 – 3ml etchant syringe; 25 – dispensing tips for etchant syringe; 50 – mixing wells; 50 – microbrushes; 1 – 8.5g syringe each of cement in shades A30, B0.5, A1 and Translucent; 30 – mixing tips (regular); 10 – mixing tips (wide) + 10 endo tips; 10 – mixing tips (wide) + 10 intra-oral tips; Technique Guide; Instructions for Use
56892	RelyX™ Ultimate Trial Kit (Translucent) 1 – 1.5ml bottle Scotchbond™ Universal Adhesive; 1 – 3ml etchant syringe; 25 – dispensing tips for etchant syringe; 50 – microbrushes; 1 – 8.5g syringe cement Translucent; 10 – mixing tips (regular); 5 – mixing tips (wide) + 5 intra-oral tips; Technique Guide; Instructions for Use
56894	RelyX™ Ultimate Trial Kit (A1) 1 – 1.5ml bottle Scotchbond™ Universal Adhesive; 1 – 3ml etchant syringe; 25 – dispensing tips for etchant syringe; 50 – microbrushes; 1 – 8.5g syringe cement (A1); 10 – mixing tips (regular); 5 – mixing tips (wide) + 5 intra-oral tips; Technique Guide; Instructions for Use



RelyX™ Ultimate Adhesive Resin Cement Introductory Kit (56891)



RelyX™ Ultimate Adhesive Resin Cement Trial Kit – A1 (56894)

Item No. Product Information

RelyX™ Ultimate Adhesive Resin Cement Refills

56887	Shade A1
56888	Shade A30 Opaque
56889	Shade B0.5 Bleach
56890	Shade Translucent
	Refills include 1 – 8.5g syringe cement; 10 – mixing tips (regular); 5 – mixing tips (wide) + 5 intra-oral tips

RelyX™ Ultimate Adhesive Resin Cement—Mixing Tips

56883	30 – mixing tips (regular)
56885	10 – mixing tips (wide) + 10 endo tips
56886	10 – mixing tips (wide) + 10 intra-oral tips

RelyX™ Try-in Paste Refills

7614A1T	A1/Light Yellow
7614A3T	A3 Opaque/Yellow Opaque
7614B0.5T	B0.5/White
7614TRT	Translucent
7614WOT	White Opaque
7614A5T	A5 Opaque/Dark
	Refills include 1 – 2g try-in paste refill

Scotchbond™ Universal Adhesive

41258	Scotchbond™ Universal Adhesive Refill 1 – 5ml adhesive vial
41263	Scotchbond™ Universal Etchant—Etching Gel 2 – 3ml syringes

Customer Care Center: 1-800-634-2249 www.3MESPE.com/RelyXUltimate



Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000
U.S.A.
1-800-634-2249

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

3M, ESPE, RelyX and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2012. All rights reserved. Multilink, OptiBond Solo Plus, Panavia and Variolink are not trademarks of 3M.

Please recycle. Printed in U.S.A.
70-2013-0504-5