# Maxcem Elite™

Self-Etch / Self-Adhesive Resin Cement

**Kerr™**

**Instructions For Use**

**Cement**

## ENGLISH

### Maxcem Elite™ Self-etch/Self-adhesive Resin Cement

#### Description

Maxcem Elite is a self-etch, self-adhesive resin cement indicated for indirect cementation of ceramic, resin and metal restorations. This product is radiopaque for easy identification in x-rays and dual-cure, allowing for maximum flexibility in its use. Maxcem Elite is packaged in a dual barrel syringe with single-use automix tips and optional curved dispensing tips to allow user to deliver desired volume of cement directly into restoration and/or tooth preparation.

#### Indications for Use

Maxcem Elite is indicated for cementation of all indirect restorations including ceramic, resin, and metal-based inlays/onlays, crowns, bridges, posts, and veneers\*. Additional indications include pit and fissure sealant, and cementation of crown restoration to implants.

\*Adhesive application on the preparation is required for veneer cementation using Maxcem Elite.

#### Contraindications

1. This product shall not be used on patients who are allergic to methacrylate resin as it may cause contact dermatitis.
2. This product shall not be placed in contact with pulp tissue. Direct exposure of this product may harm the pulp.

#### Warnings

1. Avoid contact with skin or eyes. If this product comes in contact with skin or eyes, wash thoroughly with water.

#### Precautions

1. Secure restoration with moderate pressure while cleaning excess cement.
2. Allow the cement to fully cure before finishing and polishing or adjusting occlusion to avoid potential debonding.

3. Do not use phosphoric acid to etch dentin when using Maxcem Elite in self-adhering mode, as this may cause post-operative sensitivity
4. Do not treat or clean the zirconia-based or metal-based restorations with phosphoric acid or hydrofluoric acid (HF) which can negatively impact adhesion.
5. Do not use hydrogen peroxide (H<sub>2</sub>O<sub>2</sub>) as its residues can adversely affect the setting reaction and adhesion of Maxcem Elite.
6. Do not use provisional cements containing eugenol, as these materials can inhibit the polymerization of resin cements including Maxcem Elite.
7. Use a rubber dam to keep the preparation or cavity free of any contamination during cementation.
8. Do not open foil package until initial use. Use the product within 6 months of opening the pouch but always ensure that it is not past the expiration date printed on the syringe label.
9. If prolonged contact with oral tissue occurs, flush with copious amount of water.

**Adverse reactions:**

None known.

**Instructions for Use**

**Note: Automix tips are for single patient use only to prevent cross-contamination between patients.**

Maxcem Elite is compatible with adhesives such as OptiBond™ Solo Plus, OptiBond All-In-One, or OptiBond XTR self-etch universal adhesive. Refer to adhesive instructions for use for optimal procedure. The use of other dentin and enamel adhesive system is at the discretion and sole responsibility of the dental practitioner.

**Restoration Surface Preparation**

Prepare the internal surface of the restoration according to manufacturer's instructions. Here are some general recommendations:

**Etchable Ceramic/Porcelain Restorations:** Sandblast the internal surface with 50µm alumina at a pressure of 30 psi (0.2 MPa) and etch the restoration with hydrofluoric acid. Application of the silane primer is optional.

**Metal-based, Zirconia-based, and Composite Restorations:** Sandblast the internal surface with 50µm alumina at a pressure of 60 psi (0.4 MPa) for metal-based or zirconia-based restorations, or a pressure of about 15 psi (0.1 MPa) for composite restorations.

**Resin bonded prostheses (Maryland bridges):** Etching of enamel surface with phosphoric acid gel etchant is recommended prior to cementation with Maxcem Elite.

#### **Indirect Restorations (excluding Veneers) and Implant Restoration**

1. Remove the provisional restoration.
2. Use the OptiClean rotary instrument or your choice of cleaning method to remove provisional cement and residue from the preparation. Rinse thoroughly.
3. Air dry the preparation.
4. Seat the restoration to ensure proper fit. Adjust if needed and thoroughly clean the bonding surfaces of the restoration after adjustment.
5. Select the desired shade of Maxcem Elite.
6. Remove the syringe from the foil pouch. Record the date the syringe was removed on the syringe label.

7. Remove the syringe cap. Before the first use, bleed the syringe by squeezing out a small quantity of the pastes onto a pad to equalize the base and catalyst pastes in the syringe. Secure the appropriate automix tip on the dual syringe cartridge by turning the tip clockwise. The wide 4:1 automix tips are intended for use with the intraoral tips and root-canal tips to dispense Maxcem Elite.

Dispensing of the material is done as follows:

**Inlay/Onlay** - Dispense the cement directly into the preparation covering all surfaces. Seat the restoration with moderate pressure and maintain the pressure until the cement is fully set.

**Crown** - Fill the crown with cement. Seat the restoration with moderate pressure and maintain the pressure until the cement is fully set.

**Post** - Prepare the post space. Size and fit the post. Dispense the cement onto the post or directly into the canal space using the root-canal tips. Seat and rotate the post slightly upon insertion to avoid trapping air. Allow the cement to slowly flow from canal space.

Leave the used mixing tip on the syringe as a cap until the next application.

8. When the restoration is properly seated, remove the excess cement while maintaining pressure on the restoration. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack-curing excess with a curing light for approximately 2-3 seconds, or by allowing the cement to self-cure for approximately 2-3 minutes after application or until the excess cement feels rubbery.
9. After removal of the excess cement, light cure all surfaces including margins using Demi Ultra curing light for 10 seconds. See footnote for recommended curing times for other curing light models.\*\*
10. For metal-based and opaque restorations, before proceeding to polishing and finishing, let the cement set for at least 4 minutes after seating of the restoration.
11. If needed, use finishing strips for proximal areas. Polish margins with Gloss Plus discs, cups, or points. Use HiLuster Plus polishers for a final high luster shine.
12. Verify the occlusion.

#### **Veneer Restorations**

For cementation of veneers (ideal for 1-2 units at a time), select OptiBond™ Solo Plus Adhesive or OptiBond XTR Self-Etch Universal Adhesive. Consult adhesive product instructions for use.

After adhesive is applied and cured, follow these instruction for cementation:

1. Select the desired shade of cement.
2. Remove the syringe cap. Before the first use, bleed the syringe by squeezing out a small quantity of pastes onto a pad to equalize the base and catalyst paste in the syringe. Secure the appropriate automix tip on the dual syringe cartridge by turning the tip clockwise.
3. Dispense the cement directly onto the internal surface of the veneer. Place the veneer gently onto the tooth, allowing the cement to slowly flow from all margins.
4. Tack cure the veneer into place on the facial surface, away from the margins, using a small diameter light guide for 2-3 seconds.
5. After removing the excess cement, light cure all surfaces using a Demi Ultra curing light for 10 seconds.\*\*

**Note: Maxcem Elite minimum working time is 1 minute 20 seconds at room temperature; Set time is less than 4 minutes at oral temperature. These times may vary based on storage conditions, temperature, humidity, and age of the product.**

**\*\* Recommended Curing Times:**

**Demi Ultra, Demi Plus/Demi, and L.E.Demetron II: 10 seconds.**

**L.E.Demetron I & Optilux 501: 20 seconds.**

**For all other lights, see manufacturer's recommendations.**

**STORAGE:** Store the product at ambient temperature. Avoid excess heat. For optimum freshness, store product in a refrigerator. Allow the product to reach room temperature prior to use.

Keep product in foil package until initial use. Once the foil pouch is opened, use within 6 months and prior to expiration date printed on syringe label.

“OptiBond” is a trademark owned by Kerr Corporation.

“OptiBond” is registered with the U.S. PTO and other countries.

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Kerr Italia S.r.l.